



Hood Instructions

The hood included in this kit differs somewhat from that shown in the instructional video. Either hood is acceptable for use to conduct Qualitative Fit Testing as described in the video. Please use the instructions below in donning the hood. All remaining aspects of the fit test protocol remain the same.

1. Locate the two snaps at the top of the yellow hood.
Snap them together to form a conical shaped hood.
2. Place the hood over the test subjects head.
3. Adjust hood so that that the hole is located in front of the nose and mouth.
4. Pull the drawstrings closed. The drawstrings should be tightened enough to secure the hood yet still allow room for the subject to turn his or her head once the respirator is donned.
5. Continue with fit test as described in the instructional video.